


Quick Use Guide

- ✧ Place your finger inside the clip or wrap the cuff around the foot.

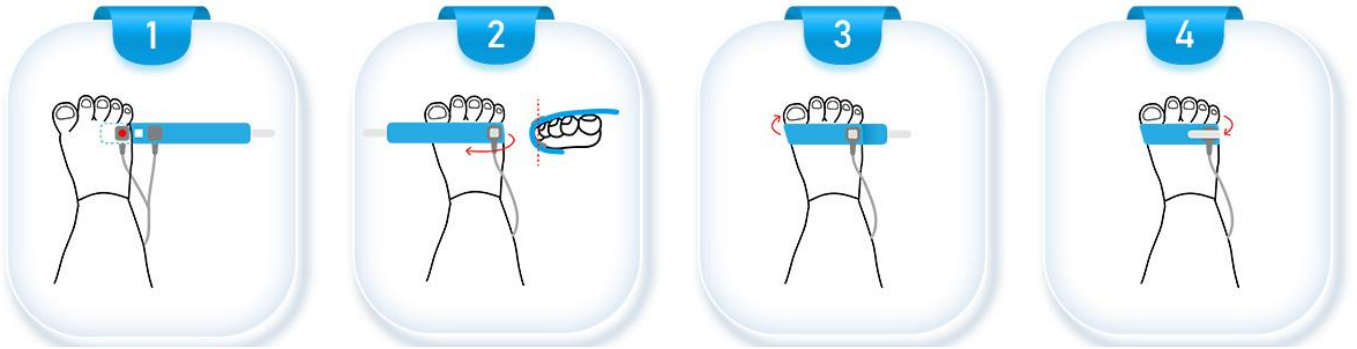

Foot Wrap Probe – Operating Diagram

- ✧ Press and hold the power button to turn it on.
- ✧ Open the app. Wait for the upper- left corner to switch from "Scanning" to "Connected".
(Omit this step if the app is not in use.)
- ✧ Remain still.
- ✧ Wait for the reading to stabilize, then take the reading.
- ✧ Auto power-off: Remove the measurement site from the probe; the device will automatically turn off after 2 minutes.

Note: Please ensure the measuring area is still, dry, warm, and free from long nails, artificial nails, or nail polish when taking a measurement.

Button Functions

▲ / ▼: Navigate up / down;

PULSE/+: Pulse sound on/off; the value of selected item + 1;

 /-: Alarm sound on/off; the value of selected item - 1 ;

MENU/RETURN: Enter the menu; return to the previous page;

 : Press and hold the power button to turn the device on or off.

Menu Functions

➤ Trend Review:

View and delete data stored on the device.

➤ **Alarm Setting:**

“SpO₂ Alarm Level” / “Pulse Alarm Level”: Enable/disable the monitoring alarm for SpO₂ and pulse rate. When enabled, an alert will be triggered if the measured value goes beyond the set range.

“SpO₂ Upper Limit” / “SpO₂ Lower Limit” / “Pulse Upper Limit” / “Pulse Lower Limit”: Set the maximum and minimum limits for SpO₂ and pulse rate.

“FingerOutAlarmSet”: Enable/disable the probe- off alarm. When enabled, a beeping alarm will sound if the probe detaches during use.

➤ **System Setting:**

- a. Set display brightness.
- b. Set alarm volume.
- c. Set pulse sound volume.
- d. Set keypad tone volume.
- e. Set data saving interval.
- f. Set language.

➤ **Date & Time Setting:**

Set the time displayed on the device home screen.